

# Citrix/Terminal Server Install directions for Exercise Pro Software

## Single Server:

1. Install Exercise Pro from the CD on the server in Install Mode
2. Download and install the patch from [www.bioexsystems.com/support.htm](http://www.bioexsystems.com/support.htm)
3. Make sure all users have full rights in the directory where Exercise Pro is installed (defaults to c:\program files\bioex)
4. Have a user start the program.
5. Register the program
6. Go to File/Settings and push the Add button in the Network section. Enter the network code.

## Multiple Servers:

1. Install Exercise Pro on any machine. Copy the expro6 directory to the file server. Uninstall Exercise Pro from this machine.
2. Make sure all users have full rights on the expro6 directory on the file server.
3. Install Exercise Pro from the CD on the server in Install Mode. Make sure to select the **Network Client** option in the install program and point at the expro6 directory on the file server.
4. Have a user start the program
5. Register the program
6. Go to the File/Settings and push the Add button in the Network section. Enter the network code.
7. For additional servers just repeat step 3.

**Note:** Unless otherwise noted in your license letter this software is licensed to be used in only one physical location. Any location further than a 15 minute walk is defined as another physical location. The location is the location of the users not the location of the server.